Contour plus

Blood Glucose Monitoring System

To check meter online registration availability, visit www.diabetes.ascensia.com/registration

NO CODING

QUICK REFERENCE GUIDE

Contour plus

System til måling af blodsukker

Kontrollér apparatets mulighed for onlineregistrering på www.diabetes.ascensia.com/registration

INGEN KODNING

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EN

Your CONTOUR PLUS meter

Use only the CONTOUR PLUS test strip

Grey End: Insert this end into the test strip port.

Sample Tip: Blood sample pulled in here.

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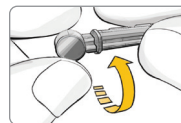
See your CONTOUR PLUS user guide and lancing device insert for complete instructions.

WARNING: Potential Biohazard

- Always wash your hands with soap and water and dry them well before and after testing or handling the meter, lancing device, or test strips.
- The lancing device provided is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.

Basic Testing - Fingertip

1. Remove the endcap from the lancing device.
2. Loosen the round protective cap on a lancet by rotating it ¼ turn, but do not remove it.

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



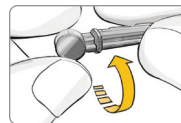
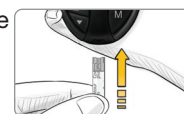
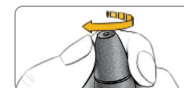
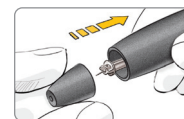
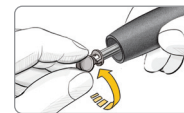
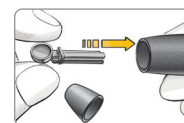
3. Insert the lancet firmly into the lancing device until it comes to a full stop.
4. Twist off the round protective lancet cap.
5. Replace the endcap.
6. Adjust the endcap dial to your preferred puncture depth setting.
7. Insert the grey end of the test strip into test strip port on the meter. The meter will turn on and a test strip with a flashing blood drop will appear on your meter display. The meter is now ready to test.

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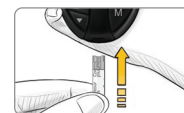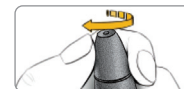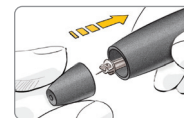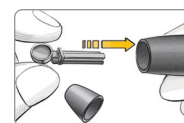
8. Press the lancing device firmly against the puncture site and press the release button.


Stroke your hand and finger toward the puncture site to form a drop of blood.

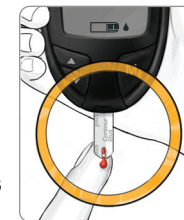9. Immediately touch the tip of the test strip to the drop of blood. **Hold the strip in the blood drop until the meter beeps.** If the first blood drop is not enough, the meter may beep twice. You have about 30 seconds to apply more blood to the same strip. Your result is displayed after a 5-second countdown.

Fingertip Testing

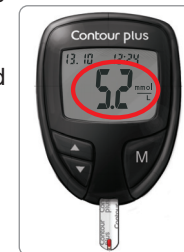10. To turn your meter off, remove the test strip.

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WARNING

Potential Biohazard

- Always dispose of used test strips and lancets as medical waste or as advised by your health care professional.
- The lancing device provided is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.

Small Parts

- Keep out of reach of children. This kit contains small parts that could cause suffocation if accidentally swallowed.
- Keep batteries away from children. Many types of batteries are poisonous. If swallowed, immediately contact your poison control center.

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WARNING: Potential Biohazard

All blood glucose measuring systems are considered biohazardous. Health care professionals or persons using this system on multiple patients should follow the infection control procedure approved by their facility for the prevention of blood-borne transmissible diseases.

Using Advanced Mode

Setting Your Meter to Advanced Mode

To set your meter to Advanced Mode, and take advantage of meal markers and other available features:

1. Press **M** to turn your meter on.
2. Press and hold ▲ or ▼ for 3 seconds.
3. L-1 will flash.
4. Press ▲ or ▼ to change to L-2.
5. Press **M** to set.

Find out how these additional features can help you manage your diabetes.

Contact Customer Service at +27 11 784 0077
www.diabetes.ascensia.com

After you perform a test and see your results, use the markers on the display:

To Mark Results >	You Will See >	To Set >	You Will See >	What It Means >	Using The Reminder
Pre-Meal Marks a test result that was taken before a meal. Press ▲ or ▼		Press M		Result is marked as a pre-meal test. Press M again to set a reminder to run a post-meal test.	 When the reminder sounds, the meter turns on, and you will see your pre-meal test result. Press any button once to silence reminder and twice to turn the meter off. Run another blood test and mark it with a 🍏 post-meal marker (see chart at left).
Post-Meal Marks a test result that was taken after a meal. Press ▲ or ▼		Press M		Result is marked as a post-meal test.	
Diary Marks any result as being unique or different in some way. Press ▲ or ▼		Press M		Result is marked in memory as unique. You should note this in your logbook.	

Sets a reminder (pre-set at 2 hours) to run a post-meal test. See user guide for complete instructions on how to use the markers and adjust the reminder times.

Error Codes and Symbols

See your user guide for a complete list.

E1	Temperature out of range.
E2	There was a problem completely filling the test strip. Retest using a new test strip.
E3	The meter is sensing a used test strip or the wrong control solution was used. Retest using a new test strip.
E11	Abnormal result. Retest using a new test strip.

Imported and Distributed by:
Pharmaco Distribution (Pty) Ltd.
91 Protea Road
Chislehurst
Sandton, 2196
Johannesburg, South Africa
+27 11 784 0077
www.diabetes.ascensia.com

Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

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Ascensia Diabetes Care Holdings AG
Peter Merian-Strasse 90
4052 Basel, Switzerland

Ascensia Diabetes Care Italy S.r.l.
Via Varesina, 162
20156 Milano, Italy

90012911
Rev. 06/24

© 2024 Ascensia Diabetes Care Holdings AG. All rights reserved. Ascensia, the Ascensia Diabetes Care logo, Clinilog, Contour, Microlet, and the No Coding logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

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